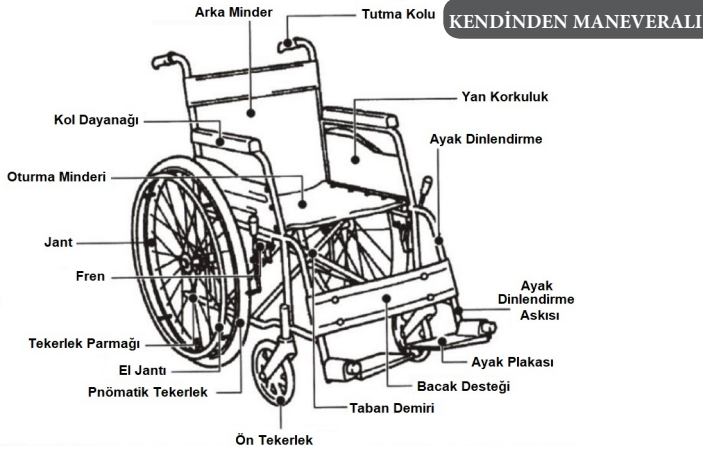
## Tekerlekli Sandalye

BME4611C

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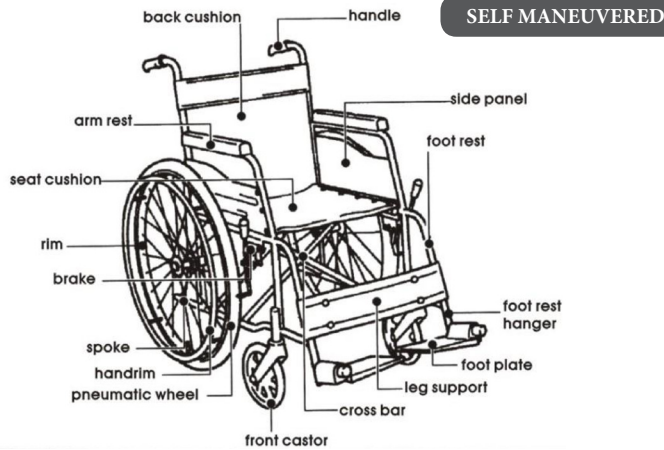
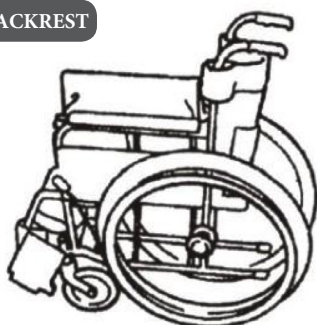
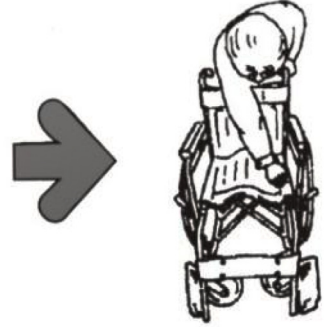
User Manual


**STEP 1**

Stand behind the wheelchair and pull it outwards to open up the chair

**STEP 2**

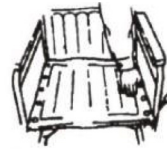Next, press the seat down to open it.

Seat pipe


**CAUTION**

Do not place your hand or finger at the seat pipe or beneath the seat


### 3. ADJUSTING THE HEIGHT OF THE FOOT PLATE

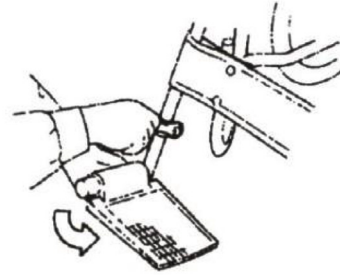**METHOD 1**

Turn the foot plate in the direction of the arrow, and adjust its height. Then tighten the nut at the side of the step pipe with the spanner provided.

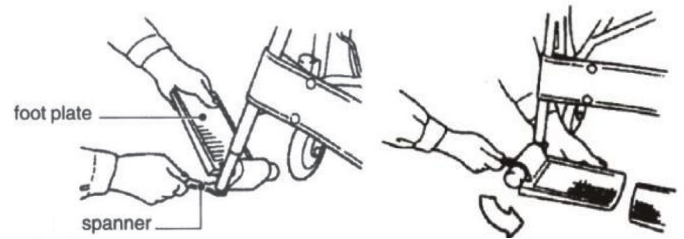**METHOD 2**


1. Using the spanner slightly loosen the portion of the step pipe where it is bolted to the foot plate


2. Turn the foot plate in the direction of the arrow. Then fix the foot plate

**CAUTION**

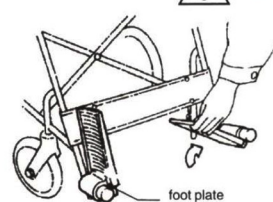
foot plate

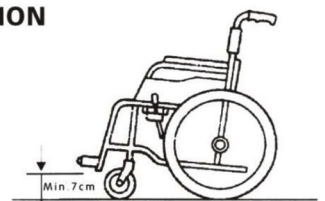Ensure that foot plate is at least 7 cm above the ground. If placed too low. The foot plate will hit obstacles when travelling on uneven ground, causing the wheelchair to be off-balance

Flip the foot plate up whenever getting on or off the wheelchair

#### 4. FOLDING THE WHEELCHAIR

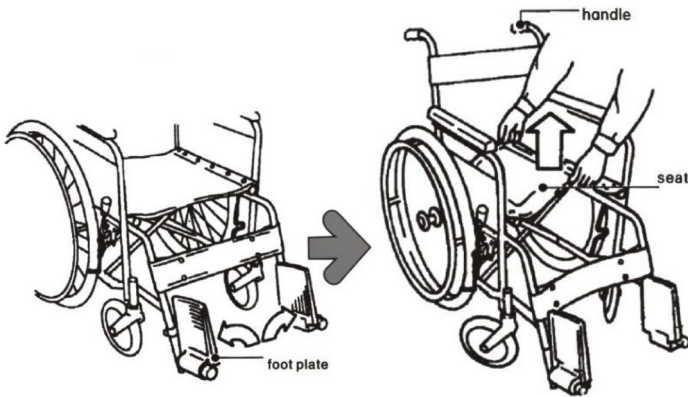
##### STEP 1

1. Flip the footplate upwards in the direction of the arrows

##### STEP 2

2. Lift the centre of the seat up to fold the wheelchair.
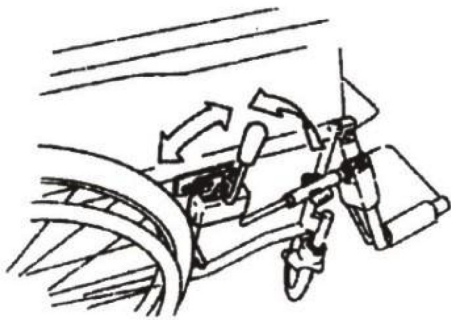
#### 5. USING THE BRAKES


Pull the brake levers on either side backwards to lock the wheelchair. To release the brake, push it forward.

#### ⚠ CAUTION

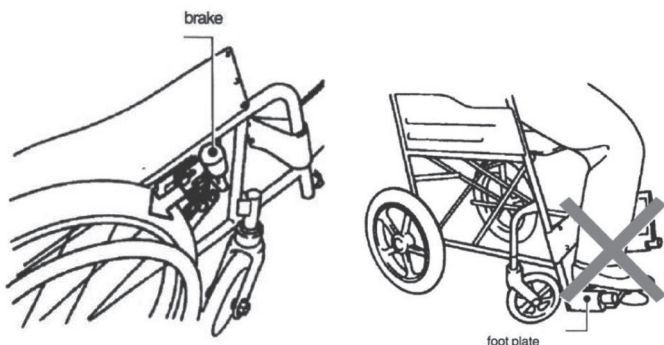1. Do remember to engage the brake even when stopping for a short while. Make it a habit to engage brakes
2. The wheelchair is unable to execute sudden stop, so do not speed while on it.
3. When turning at a sharp corner, reduce speed to avoid tipping over.
4. Before moving the wheelchair, ensure that the patient has his/her feet on the foot plate.
5. Always ensure safety first. If the patient feels unsafe being alone, he/she should ask for assistance before proceeding.



#### 6. POINTS TO NOTE WHEN USING THE WHEELCHAIR


1. Ensure that the brake is locked when getting on/off the wheelchair. Refer to page 6 on how to use the brake

2. Do not step on the foot plate when getting off the wheelchair or else the chair will tip forward.

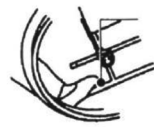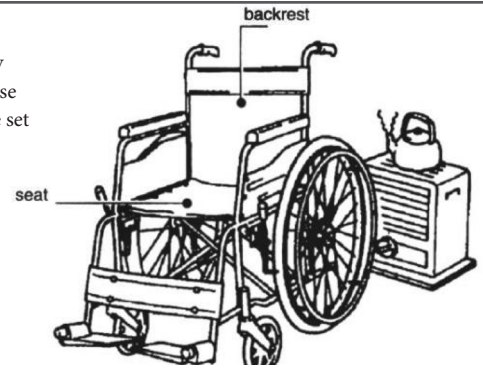3. Do not place the wheelchair near any sources of fire, or else the backrest may be set on fire, resulting in injuries.

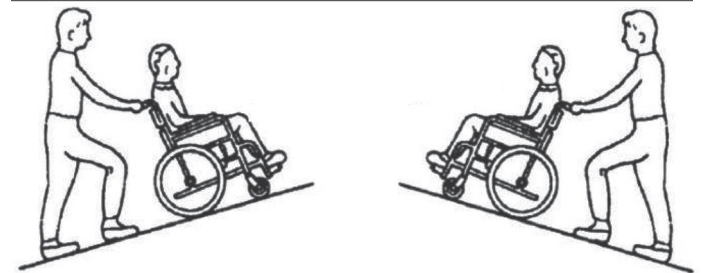4. To move up a step. The care giver pushing the wheelchair must step on the tipping bar and push downwards at the handles, such that the front wheel caster are raised onto the step. Next, pull upwards at the handles to bring up the rear wheels. Do not try to push over the steps by force.

5. to move up or down slope, ensure that a care giver is around to support the wheelchair; otherwise, the chair may overturn. For a steep downward slope, the wheelchair should be turned to face up slope. And the care giver should back the chair downwards gradually.

6. When lifting a wheelchair with the patient on it, do not hold on to the handles if the chair's backrest is the foldable kind. Otherwise, the backrest may be damaged.

Shanghai Brother Medical Products Manufacturer Co., Ltd.  
718, Fanou Modern Building, No 2449 Gonghexin Road,  
Jingan District, Shanghai City, P.R.C

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SUNGO Europe B.V.