KULLANIM KILAVUZU

TEKERLEKLİ SANDALYE

BME4634

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Shanghai Brother Medical Products Manufacturer Co., Ltd.
718, Fanou Modern Building, No 2449 Gonghexin Road,
Jingan District, Shanghai City, China

İTHALATÇI: ECE TIBBİ CİHAZLAR VE MEDİKAL SAN. TİC. A.Ş.
Maltepe Mah. Hastane Sok. No:6 Zeytinburnu/İstanbul/Türkiye
www.respirox.com.tr / www.ecemedikal.net
Tel: 444 78 96

EC REP

SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE
Amsterdam, Netherlands


Basım Tarihi: 20.12.2022

USER MANUAL

Transit Wheelchair BME4634


1. PARTS OF THE WHEELCHAIR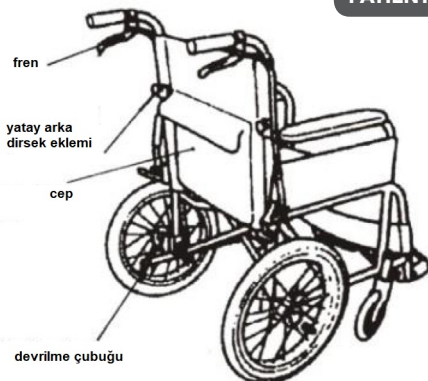

PATIENT ASSISTED

2. HOW TO UNFOLD THIS WHEELCHAIR

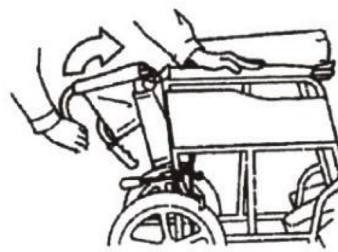STEP 1

Grasp the handle and lift upwards in the direction of the arrows

STEP 2

While lifting the handle. Push against the armrest for support

STEP 3

Stand behind the wheelchair and pull it outwards to open up the chair


STEP 4

Next, press the seat downwards to open it.


CAUTION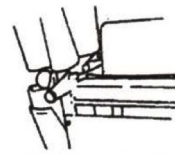

Do not place your hand or finger at the seat pipe or beneath the seat

When the pin is locked, a click sound can be heard

3. ADJUSTING THE HEIGHT OF THE FOOT PLATE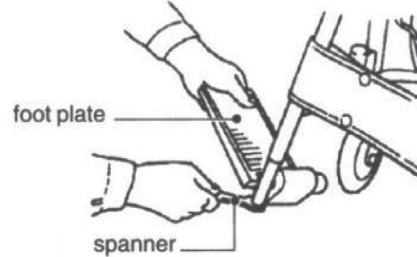


foot plate


spanner

1. Using the spanner slightly loosen the portion of the step pipe where it is bolted to the foot plate

2. Turn the foot plate in the direction of the arrow. Then fix the foot plate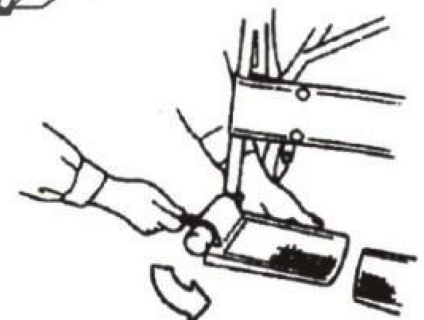


CAUTION

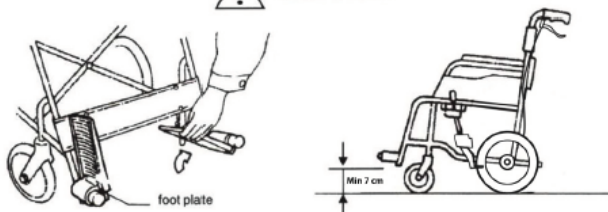Flip the foot plate up whenever getting on or off the wheelchair

Ensure that foot plate is at least 7 cm above the ground. If placed too low. The foot plate will hit obstacles when travelling on uneven ground, causing the wheelchair to be off-balance

4. FOLDING UP THE WHEELCHAIR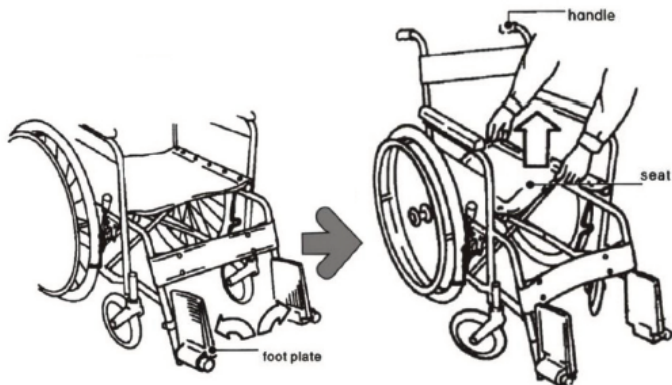


STEP 1

Flip the footplate upwards in the direction of the arrows

STEP 2

Lift the centre of the seat up to fold the wheelchair.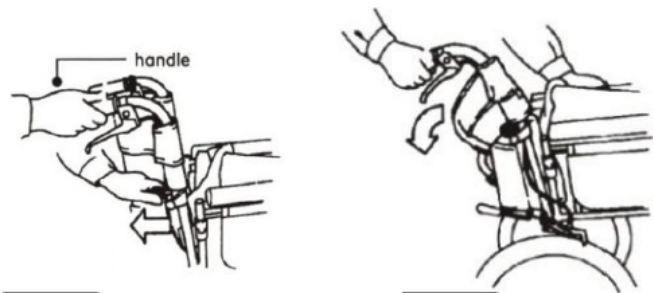

STEP 3

Hold on to the handle with one hand, and pull the knob backwards with the other.

STEP 4


The handle can now be dropped downwards

CAUTION


1. Do remember to engage the brakes even when stopping for a short while. Make it a habit to engage brakes.
2. The wheelchair is unable to come to a sudden stop, so do not speed while on it.
3. When turning at a sharp corner, reduce speed to avoid tipping over.
4. Before moving the wheelchair, ensure that the patient has his/her feet on the foot plate.
5. Do not stand or step on the footplates while transferring to or from your wheelchair. This could cause the wheelchair to tip or may cause personal injury or damage to your wheelchair.
6. Always engage wheel locks before transferring, using a wheelchair lift or using an elevator.
7. Always ensure safety first. If the patient feels unsafe being alone, he/she should ask for assistance before proceeding.

5. POINTS TO NOTE WHEN USING THE WHEELCHAIR

1. Do not place the wheelchair near any sources of fire, or else the backrest may be set on fire, resulting in injuries.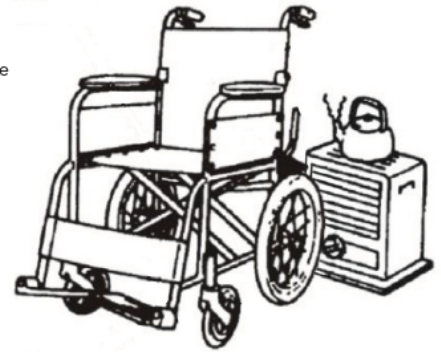

2. To move up a step. The care giver pushing the wheelchair must step on the tipping bar and push downwards at the handles, such that the front wheel caster are raised onto the step. Next, pull upwards at the handles to bring up the rear wheels. Do not try to push over the steps by force.

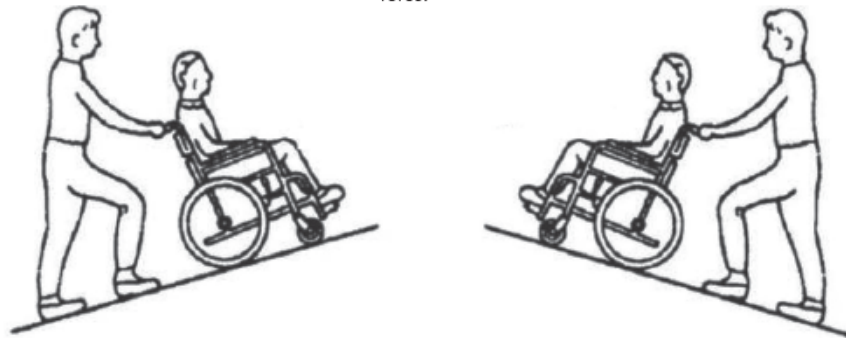3. to move up or down slope, ensure that a care giver is around to support the wheelchair; otherwise, the chair may overturn. For a steep downward slope, the wheelchair should be turned to face up slope. And the care giver should back the chair downwards gradually.

4. When lifting a wheelchair with the patient on it, do not hold on to the handles if the chair's backrest is the foldable kind. Otherwise, the backrest may be damaged.

Shanghai Brother Medical Products Manufacturer Co., Ltd.
718, Fanou Modern Building, No 2449 Gonghexin Road,
Jingan District, Shanghai City, China

IMPORTER: ECE TIBBİ CİHAZLAR VE MEDİKAL SAN. TİC. A.Ş.
Maltepe Mah. Hastane Sok. No:6 Zeytinburnu/İstanbul/Türkiye
www.respirox.com.tr / www.ecemedikal.net
Tel: 444 78 96

EC REP

SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE
Amsterdam, Netherlands